



**PHILIPS**

Ultrasound

Lumify

# Simplicity, quality, and **flexibility**

Philips Lumify ultrasound system specifications

# Contents

<b>1</b>	<b>Introduction</b>	<b>3</b>
1.1	Applications	3
	Key advantages	3
<b>2</b>	<b>System overview</b>	<b>4</b>
2.1	System architecture	4
2.2	Imaging modes	4
2.3	Image optimization	4
2.4	Touchscreen interface	4
<b>3</b>	<b>Workflow</b>	<b>5</b>
3.1	Home screen	5
3.2	Display information	5
3.3	Cineloop review	5
3.4	Exam documentation	5
3.5	Connectivity	5
<b>4</b>	<b>Transducers</b>	<b>6</b>
4.1	Transducer application guide	6
	Curved array	6
	Linear array	6
	Sector array	6
<b>5</b>	<b>Physical specifications</b>	<b>7</b>
	Localization options	7
	Training and user documentation	7
	Electrical safety standards	7
	Agency approvals	7



# 1. Introduction

Get exceptional quality ultrasound imaging on your compatible smart device with Lumify, the newest app-based ultrasound solution from Philips. It's a dependable and comprehensive solution that brings simplicity, mobility, and flexibility to point-of-care ultrasound technology.

## 1.1 Applications

- Abdominal
- Obstetrical/Gynecological
- Gallbladder
- Lung
- Soft tissue
- Vascular
- Echo
- FAST
- Musculoskeletal (MSK)
- Superficial

### Key advantages

- Combines the exceptional quality of Philips imaging technology with the mobility and connectivity of a compatible smart device
- Offers app-based ultrasound with flexible subscription pricing
- Provides an intuitive interface to help guide decision-making for a confident treatment plan



# 2. System overview

## 2.1 System architecture

---

- Next-generation micro-digital broadband beamformer
- Microfine 2D focusing with dynamic focal tuning
- Dynamic range up to 170 dB (full-time input)
- 65,536 digitally processed channels
- SonoCT real-time beam-steered compound imaging
- XRES adaptive image processing
- AutoSCAN: no-touch continuous intelligent optimization for 2D
- Gray shades: 256 (8 bits) in 2D
- Acquisition frame rate: up to 79 frames per second in high frame rate mode (dependent on field of view, depth, and angle)
- Power save mode that automatically senses when you are not actively scanning and reduces frame rate
- Harmonic imaging

## 2.2 Imaging modes

---

- Philips microfine 2D focusing
- Color flow

### 2D mode

- Microfine 2D focusing
- AutoSCAN
- Digital reconstructed zoom up to three times with pan capability with intuitive multi-touch control
- Cineloop image review (3 to 10 seconds loop length)
- 256 (8 bits) discrete gray levels Philips microfine 2D focusing
- Intuitive “Pinch” to zoom and “Touch” to pan image
- Full-screen mode
  - Available in live-imaging or review

### Color Doppler

- Gain 0 to 100 in steps of one
- Cineloop review
- Velocity display
- Touch-controlled color Region of Interest: size and position
- Touch-controlled color steering
- Maps, filters, color sensitivity, scale, line density, smoothing, echo write priority, color persistence, gain, and baseline optimized automatically by preset

### M-mode

- Available on all transducers
- Time markers: 0.2 seconds

## 2.3 Image optimization

---

### SonoCT real-time compound imaging

- High precision beam-steered image compounding acquires additional tissue imaging information compared to orthogonal beams and reduces angle-generated artifacts
- Enhanced needle visualization
- Multiple beam-steered lines of sight
- Operates in conjunction with harmonic and XRES imaging

### Tissue Harmonic Imaging

- System processing of second harmonic frequencies (nonlinear energy) in tissue
- Extends high performance imaging capabilities to most patient body types
- Available in 2D imaging mode
- Image display with reduced artifacts

### XRES adaptive imaging processing

- Enhances images without altering the image resolution
- Reduces artifacts, enhances contrast resolution, visibility of tissue texture patterns, and border definition
- Available in 2D, zoom, post-freeze, and when capturing loops
- Applied to grayscale data of 2D images

### AutoSCAN intelligent optimization

- No-touch continuous intelligent optimization
- In 2D mode, automatically identifies tissue type and continuously adjusts TGC and receiver gain to achieve tissue uniformity and brightness
- Full-screen mode
  - Available in live-imaging or review

## 2.4 Touchscreen interface

---

- Multi-touch user interface
- Alphanumeric QWERTY soft keyboard
- Imaging mode keys: 2D and color Doppler
- 2D image controls: depth, freeze, gain, and power
- Depth to 30 cm (exam-specific)
- Measurements: 2D distance calculation; M-mode calculation
- Color Doppler controls: angle, scale (fast/slow flow), gain, and volume
- Image acquisition keys: review, save image, and save loop
- Annotation controls: text and erase



# 3. Workflow

## 3.1 Home screen

- Simplified home screen for quick access to scan, create patient profile, and select presets
  - Four clinical application presets (C5-2)
  - Five clinical application presets (L12-4)
- Main menu

## 3.2 Display information

- On-screen display of all pertinent imaging parameters for complete documentation, including transducer type and frequency range, active clinical options and optimized presets, display depth, grayscale, color map, color scale, frame rate, 2D gain, color gain, color image mode, and patient name
- Depth to 30 cm (exam-specific)
- Real-time display of Mechanical Index (MI)
- Real-time display of Thermal Index (TIb, TIc, TIs)
- Annotation text – places, moves, erases, modifies, or appends typed text and arrows
- Annotation erased with start of new study
- On-display centerline marker aligned with transducer centerline marker
- End Exam – closes study and returns user to home screen for efficient workflow
- Network connectivity icon to allow immediate feedback about network condition
- Battery status icon and warning to allow immediate feedback about battery condition (depending on compatible smart device chosen)

## 3.3 Cineloop review

- Acquisition, storage in memory, and display in real time of up to 10 seconds of 2D and color images
- Images for retrospective review and image selection
- Slide control of frame-by-frame image selection
- Functions in 2D and color Doppler imaging modes

## 3.4 Exam documentation

- Input and output ports (depending on compatible smart device chosen)
  - USB port on smart device; uses include connecting the transducer, supporting data transfer, and charging
  - Some devices include video output\*
  - Wi-Fi/cellular; uses include DICOM networking, emailing exams, and network shared drive connection for EMR

\*Specific capabilities such as internal storage size, ports, video connection, and cellular connectivity depend on the specific user-selected host smart device.

## 3.5 Connectivity

- Patient data storage on device
- Configurable barcode reader software utilizing device camera
- DICOM modality worklist (query retrieve)
- Direct digital storage of single-frame color and B/W images to internal hard disk
- Direct digital storage of B/W and color loops to internal hard disk
- Ability to export in PC format (MP4 clips, PNG images) via email or direct connection to PC
- Extensive image management capability, including thumbnail image review
- Exam directory
- DICOM image store
- Export to network share drive
- User may email patient exams
- Option to configure patient data in the DICOM header and images (not DICOM tags), as well as anonymize PC format images for exported images and loops
- Use Google Cast to mirror device screen to secondary display



# 4. Transducers

## 4.1 Transducer application guide



Transducer		C5-2	L12-4	S4-1
Type of array		Curved	Linear	Sector
Application				
Abdominal	0-4 cm	•		•
	5-10 cm	•		•
	> 11 cm	•		•
	Gallbladder	•		•
	Lung	•	•	•
Gynecology	Transabdominal < 10 cm	•		•
	Transabdominal > 11 cm	•		•
Obstetrics	1st trimester 10-12 cm	•		•
	2nd trimester 12-18 cm	•		•
	3rd trimester 15-20 cm	•		•
Echo				•
Vascular	0-3 cm	•		
	3-8 cm	•		
Musculoskeletal			•	
Superficial			•	
Vascular			•	
Soft tissue			•	
FAST				•



## Curved array

### C5-2 broadband curved array

- Automatically optimized focal zone based on preset focal zone
- Continuous dynamic receive focusing
- 128 elements
- 5 to 2 MHz extended operating frequency range
- 67.5° field of view
- High-resolution imaging for abdomen, gallbladder, lung, and Ob/Gyn applications
- Supports 2D, color Doppler, and Tissue Harmonic Imaging
- Lightweight replaceable USB cable

## Linear array

### L12-4 broadband linear array

- Automatically adjusted focal zone based on preset focal zone
- Continuous dynamic receive focusing
- 128 elements
- 4 to 12 MHz extended operating frequency range
- 34.5 mm field of view
- High resolution imaging for lung, musculoskeletal, soft tissue, superficial, and vascular applications
- Supports 2D, color Doppler, and Tissue Harmonic Imaging
- Lightweight replaceable USB cable

## Sector array

### S4-1 broadband sector/phased array

- Automatically adjusted focal zone based on preset focal zone
- Continuous dynamic receive focusing
- 64 elements
- 1 to 4 MHz extended operating frequency range
- 90° field of view
- High resolution imaging for lung, echo, abdominal, Ob/Gyn, FAST applications
- Supports 2D, color Doppler, and Tissue Harmonic Imaging
- Lightweight replaceable USB cable

# 5. Physical specifications



## Localization options

Software – English

## Training and user documentation

English

## Electrical safety standards

- IEC 60601-1, Medical Electrical Equipment: General requirements for safety, including all applicable collateral and particular standards, as well as all applicable deviations
- IEC 60601-1-2, Collateral Standard, Electromagnetic compatibility – requirements and test
- IEC 60601-2-37, Particular Requirements for the basic safety and essential performance of ultrasonic medical diagnostic and monitoring equipment
- ANSI/AAMI ES60601-1, Medical Electrical Equipment: General requirements for basic safety and essential performance

See Lumify portal for the list of compatible smart device options: [www.philips.com/lumify](http://www.philips.com/lumify).

## Environmental standards

- Home Healthcare standard (60601-1-11)
- EMT standard (60101-1-12)
- Military standard for helicopter – RTCA DO-160G

## Agency approvals

- CE Mark in accordance with the European Medical Device Directive issued by British Standards Institute (BSI)



